



# 101M2/M203

## MODIFIED LAY DOWN VEST

### IMPORTANT INFO

- Use for body casts, hip spica casts, braces and any other configuration where the passenger must be transported laying down.
- M203 for passengers over 12 months old, weighing 22 – 75 lbs.
- 101M2 for passengers weighing 60 – 106 lbs.
- A layer of padding no thicker than 2” may be used under the passenger for added comfort.
- Can be secured by using lap type seat belts or shoulder/lap belts. No mount is needed.
- This is also a great product for sleeping children.
- REGISTER this transportation vest to be notified of a safety recall.

### WARRANTY

*EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.*


### WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

### PINCH TEST PROCEDURE

#### THE PINCH TEST

Tighten vest so that no pinch in the webbing can be achieved on the chest strap.  
When vest is snug fingers will be unable to grab any webbing.

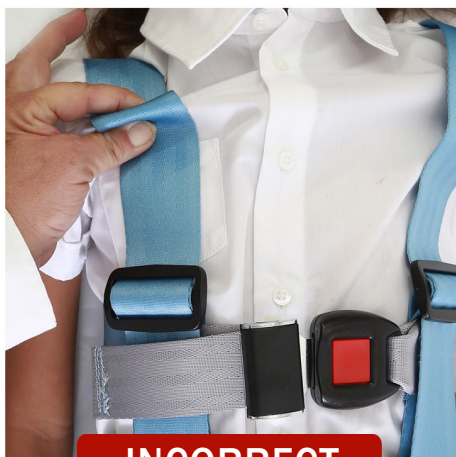

**INCORRECT**



**CORRECT**



# 101M2/M203

## MODIFIED LAY DOWN VEST

## INSTALLATION INSTRUCTIONS

### TO SECURE VEST ONTO PASSENGER

1. Place vest around passenger with E-Z-ON parachute logo on the front.
2. Close and secure push buttons.
3. Bring crotch straps through passenger's legs and secure.
4. Pull downward on strap to adjust. Pull webbing snug but not too tight. Make sure to position the hip strap low, and over hip bones.
5. Pull loose end of shoulder strap through bar slide.
6. Hold bar slide with one hand while pulling loose end to lengthen or shorten strap.  
*\*Do not pull too tight, as this will cause vest to pull off hips.*
7. Back thread to lock in place.

### PINCH TEST

Please refer to procedure on reverse side.

### TO SECURE PASSENGER TO REAR SEAT

NOTE: Rear car seat must be flat.

8. Lay passenger in center of rear seat away from doors of the vehicle. They must be able to fit on rear seat laying lengthwise.
9. For children over 30" tall route the vehicle seatbelt close to the passenger's head through the chest strap, buckle and tighten.  
*\*For passenger under 30" tall route seatbelt through the chest and hip strap, buckle and tighten.*
10. Route the second seat belt through the hip strap, buckle and tighten.
11. Secure seat belts as tightly as possible.
12. Place pillows behind passenger's head and neck for comfort and protection.

### USE OF EXTREMITY BELT

If 1 leg is casted, be sure that the cast is placed against the rear seat. If both legs are cast, place the extremity belt around both casts at the knee area.

13. Place extremity belt around casted leg(s) at the knee(s).
14. Route seatbelt through extremity belt, buckle and tighten.
15. Secure extremity belt by pulling it snug.

